



# Standard Operating Procedures



220 Mount Laurel Road  
Mount Laurel, NJ 08054  
p: 856-234-7626  
f: 856-231-8393

Name of Policy:	How to Use My Pet Laser for Laser Treatments	Departments:	Primary Care
Category & No.:	PAT 20.1	Effective By:	8/9/2025
Version No.:	1.0	Revision History:	Original

**Purpose:** Manage pain, reduce inflammation and accelerate healing for various conditions

**Prerequisite(s):** goggles, away from reflective surfaces, my pet laser (located in pc tx)

**Background Information:** The My Pet Laser is a 15,000 mw laser that helps promote healing and can be used to treat acute or chronic injuries (ie: sprains, strains, musculoskeletal abnormalities, wounds, skin conditions, etc)

## Procedure:

### Level 1 -

Effect - stimulatory

Depth - deeper

Dose - low

Preset time - 1 minute

Purpose - reduces inflammation, improves circulation, promotes healing

### Level 2 -

Effect - stimulatory

Depth - deep

Dose - low

Preset time - 1 minute

Purpose - promotes healing of bone, muscle, tendon and ligament injuries, reduces arthritic pain, stimulates bone therapy

### Level 3 -

Effect - inhibitory

Depth - superficial

Dose - high

Preset time - 3 minutes

Purpose - reduces edema, promotes pain relief

### Level 4 -

Effect - stimulatory

Depth - variable

Dose - low

Preset time - 2 minutes

Purpose - injuries, trauma, surgery sites within 48 hours

Level 5 -

Effect - inhibitory

Depth - superficial

Dose - high

Preset time - 5 minutes

Purpose - pain control

**References:**

<https://vet.multiradiance.com/vet/products/my-pet-laser-2.0.1>

And manual located in carry case with laser